




PRODUCT PROFILE

L.A. Ez-Line

Product	Dental Unit Water Line Cleaner for independent water bottle systems.
Description	L.A. Ez-Line reduces the risk of cross-contamination, bacterial aerosols and microbial contamination while cleaning dental unit tubing.
Packaging	480 mL or 10 L bottle.

Features	Benefits
<i>Easy to Use</i>	L.A. Ez-Line is supplied ready-to-use, no mixing required.
<i>Cost Effective</i>	Approximately 15 mL (½ ounce) of L.A. Ez-Line is required per dental unit water line.
<i>Time Efficient & Economical</i>	Run L.A Ez-Line through the dental lines at the end of the day. Let it sit overnight. In the morning, re-attach water for daily use and run water for 1-2 minutes to remove L.A Ez-Line from the lines.
<i>Non-toxic</i>	Contains no designated toxic substances..
<i>Pleasant Scent</i>	Mint scent is clean and refreshing.
<i>Effective</i>	L.A Ez-Line reduces the risk of cross-contamination, bacterial aerosols and microbial contamination in dental unit waterlines.
<i>Non-corrosive & Non-staining</i>	Will not corrode metals, clog valves, stain or adhere to lines.

L.A. products contain Lactic Acid.
 Lactic Acid is produced by fermentation of carbohydrates. It is used in cosmetic, food and detergent industries.
Lactic Acid is an organic renewable resource.
Lactic Acid has anti-bacterial properties.
Lactic Acid is effective and economical.



If you have any questions about **L.A. Ez-Line**, contact your local area sales representative or call **GERMIPHENE** Corporation toll-free at 1.800.265.9931
 Website: www.germiphene.com Email: info@germiphene.com

