




# Product Profile

## GERMIPHENE NEUTRAL DETERGENT

A neutral, non-ionic detergent solution for surface and pre-disinfection cleaning. Neutral Detergent will not deactivate disinfectants or chemical sterilants.

<b>Composition</b>	A non-ionic surfactant combination in aqueous solution.
<b>Effective</b>	Neutral Detergent is a powerful, non-ionic detergent with a fresh fragrance that rapidly and efficiently enables the removal of biological soiling (removal of blood, bodily fluids and tissue fragments) from clinical surfaces and surgical instruments and equipment (including dental handpieces) prior to disinfection or sterilisation.
	Not to be mixed with any other detergents or cleaning agents.
	Neutral Detergent can be used for general cleaning.
<b>Safe</b>	Proprietary formulation leaves surfaces resistant to resoiling.
	Non-toxic formula, with neutral pH is mild on surfaces and instruments.
<b>Convenient</b>	No mixing required. Ready-to-use formula.
	Streak-free. No rinsing required.
	Multi-use formula for surfaces and surgical instruments.
<b>Directions</b>	Easy-to-use. Simply spray and wipe.
	<b>SURFACES:</b> Spray or wipe on with a clean cloth undiluted Neutral Detergent onto the surface to be cleaned. Wipe surface as required until all visible biological soiling has been removed. Surface can be rinsed or wiped clean or disinfectant solution applied.
<b>Directions</b>	<b>SURGICAL INSTRUMENTS:</b> Spray or wipe on with a clean cloth undiluted Neutral Detergent onto the instrument. Clean instrument with a clean cloth until all visible biological soiling has been removed. Instrument can now be rinsed clean in preparation for effective disinfection or sterilisation as required.
<b>Fragrance</b>	 Laundry Fresh fragrance
<b>Available Sizes</b>	 4L  700 mL (empty dispensing bottle)



For more information contact your district sales manager or call Germiphene 1.800.265.9931. Visit us online at [www.germiphene.com](http://www.germiphene.com) or send email inquiries to: [info@germiphene.com](mailto:info@germiphene.com)