



# PRODUCT PROFILE

## TRÉS CLEAN

<b>Product</b>	Tray and Instrument Cleaner
<b>Description</b>	A concentrated, non-rusting and biodegradable cleaner for use on dental trays and/or instruments.
<b>Function</b>	<b>TRÉS CLEAN</b> will loosen any remaining traces of alginates, zinc-oxide eugenol, carboxylate and phosphate cement and adhesive varnish from impression trays and instruments.
<b>Packaging</b>	1 litre concentrate

Features	Benefits
<i>Cost Effective</i>	Concentrated form should be diluted to make a 5 to 10% solution (dilute 50-100 mL in 1 litre of water). 1 litre can make up to 20 litres of use solution.
<i>Non-Rusting</i>	Kind to all plastic and non-corroding metal impression trays and instruments.
<i>Long Lasting</i>	Solution can be used for 2 to 3 days.
<i>Easy to Use</i>	After pre-cleaning, place instruments and upright trays in the solution. Allow to soak for 2-3 hours. Hot water accelerates the cleaning process. Deposits will be softened and easily removed either under running water or with a brush. For best results, soak overnight.
<i>Biodegradable</i>	<b>TRÉS CLEAN</b> is biodegradable and very mild in its cleaning action.

If you have any questions about **TRÉS CLEAN**, contact your local area sales representative or call **GERMIPHENE** Corporation toll-free at 1.800.265.9931  
Website: [www.germiphene.com](http://www.germiphene.com) E-mail: [admin@germiphene.com](mailto:admin@germiphene.com)

