steriSAFE

Congratulations on your subscription to the steriSAFE program by Germiphene Corporation. Enclosed are 26 tests, each containing two biological spore strips. These strips are inoculated with highly resistant, but not pathogenic organisms. Keep them at room temperature and away from moisture until they are exposed.

*DIRECTIONS FOR USE:

Please DO NOT REMOVE the **TEST**, and the **CONTROL** strips from the blue tyveck packaging.

1. Place one strip labeled **TEST** in the sterilizer along with a normal load.
2. Expose/Process **TEST** strip in normal sterilization cycle.
 - A. The strip should be placed in the most difficult to sterilize location in the sterilizer, such as the front and middle-back of an autoclave.
 - B. To assure proper circulation between the loads, do not overload the sterilizer.
 - C. The parameters for the sterilization cycle will depend upon the sterilizer and the amount and type of material. Usually 250°F at 15 (psig) pressure, for a minimum of 30 minutes. Do not use the strip when the sterilizer is performing a cleaning cycle.
 - D. The decontamination of biomedical waste regularly requires 60 minutes at 250°F.
 - E. Proper operating procedure and settings may be contained in the unit specific operation manual.
3. The second strip labeled **CONTROL** should not be exposed/processed.
4. Once the Sterilization Cycle is complete, place the **TEST** and **CONTROL** strips, and the completed Sterilization Monitoring Service Report Form into the self-addressed envelope and mail to Germiphene Corporation. The Report Form can be downloaded online at www.Germiphene.com. You can fill in the form electronically or by hand. It must be completed in its entirety, excluding the sections marked **FOR LAB USE ONLY**. ***Please remember to provide your e-mail address on the Report Form in order to receive your test results!**
5. The strips will be cultured for 24 hours and a report will be sent to you via e-mail.
6. If the test shows any growth of the indicator organism, Germiphene Corporation will notify your office immediately by e-mail.
7. Germiphene Corporation will recommend appropriate equipment checks or corrective procedures.
8. If for any reason the integrity of the sterilization process is in question, the load should be considered contaminated and reprocessed.

Please contact us if you have any questions regarding sterilization or sterilization monitoring.

* **Do NOT process the CONTROL STRIP (red text on strip)!**

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